

Conveen® Optima

Measurements are close approximations.

40
Gray

35
Blue

30
Green

28
Purple

25
Yellow

21
Red

To determine circumference: Measure the penis at the middle of the shaft when it is flaccid (not erect). Place the penis in the arcs to determine the best fit.

If you are between two sizes (both length and circumference), select the smaller size.

Measure length of penis by placing the end of the blue strip guide (short or standard) along side the penis beginning at the base (point closest to the body) and measure to the tip of the penis.



The Coloplast logo is a registered trademark of Coloplast A/S.
© 2015 Coloplast Corp. All rights reserved.
M1675N 05.15

base

Short

tip

tip

Standard

base