

Conveen® Optima

Measurements are close approximations.

40  
Gray

35  
Blue

30  
Green

28  
Purple

25  
Yellow

21  
Red

**To determine circumference:** Measure the penis at the middle of the shaft when it is flaccid (not erect). Place the penis in the arcs to determine the best fit.

**If you are between two sizes (both length and circumference), select the smaller size.**

**Measure length of penis** by placing the end of the blue strip guide (short or standard) along side the penis beginning at the base (point closest to the body) and measure to the tip of the penis.



The Coloplast logo is a registered trademark of Coloplast A/S.  
© 2015 Coloplast Corp. All rights reserved.  
M1675N 05.15

base

Sport

tip

tip

Standard

base